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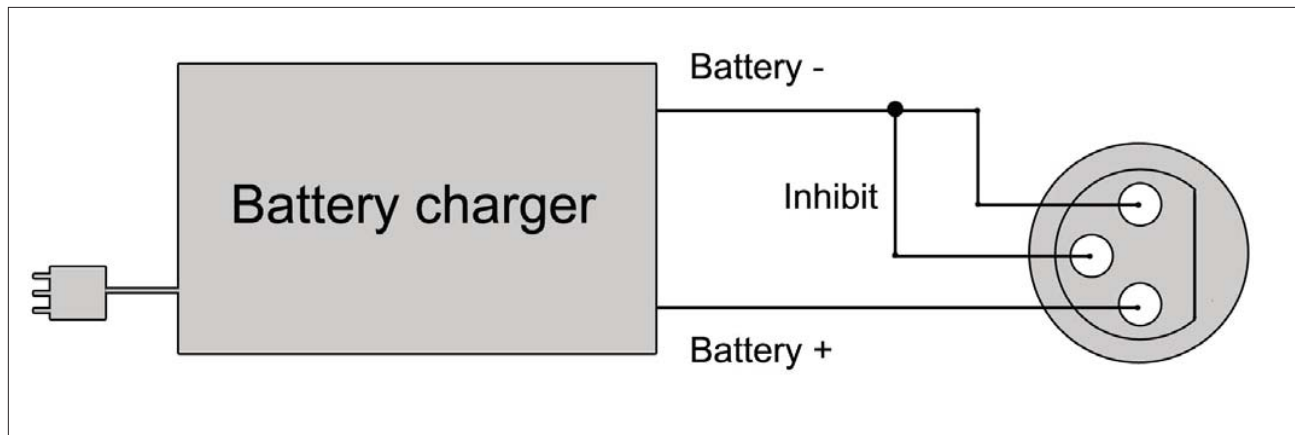
© 2011 Handicare

All rights reserved.

The information provided herein may not be reproduced and/or published in any form, by print, photoprint, microfilm or any other means whatsoever (electronically or mechanically) without the prior written authorization of Handicare.

The information provided is based on general data concerning the construction known at the time of the publication of this manual. Handicare executes a policy of continuous improvement and reserves the right to changes and modifications.

The information provided is valid for the product in its standard version. Handicare can therefore not be held liable for any damage resulting from specifications of the product deviating from the standard configuration. Illustrations contained in the manual may deviate from the configuration of your product.

The information made available has been prepared with all possible diligence, but Handicare cannot be held liable for any errors contained in the information or the consequences thereof.

Handicare accepts no liability for loss resulting from work executed by third parties.

Names, trade names, trademarks etc. used by Handicare may not, as per the legislation concerning the protection of trade names, be considered as being available.

2011-07

|     |                                             |    |
|-----|---------------------------------------------|----|
| 1   | Introduction                                | 19 |
| 1.1 | This user manual                            | 19 |
| 2.  | The controller                              | 20 |
| 2.1 | DX Rem 48 controller                        | 20 |
| 3   | Driving the wheelchair with the controller  | 22 |
| 3.1 | Switching the controller on or off          | 22 |
| 3.2 | Driving the wheelchair                      | 22 |
| 3.3 | Speed                                       | 22 |
| 4   | Operating the electrical adjustment options | 23 |
| 4.1 | Selecting the desired adjustment option     | 23 |
| 5   | Lights                                      | 24 |
| 6.  | Troubleshooting                             | 25 |
| 7.  | Locking the controller                      | 26 |
| 8   | Technical specifications                    | 27 |
| 9.  | Technical diagrams                          | 28 |
| 9.1 | Technical diagram                           | 28 |
| 9.2 | Technical diagram battery charger           | 29 |

# 1 Introduction

## 1.1 This user manual

This user manual will help you to use and maintain the controller of your power wheelchair safely. This user manual is a supplement to Handicare's general wheelchair user manual. When necessary this user manual refers to other manuals as shown below:

Wheelchair: Refers to the general wheelchair user manual.

Battery charger: Refers to the user manual for the battery charger.

**Read this user manual and the other user manuals referred to carefully before using the product.** If one of the user manuals was not included with your wheelchair, please contact your dealer immediately.

In addition to this user manual, there is also a service manual for qualified specialists.

CONTACT HANDICARE IF YOU HAVE A VISUAL IMPAIRMENT.

## 2. The controller

### 2.1 DX Rem 48 controller

A controller will usually have three basic functions:

- Driving and steering a wheelchair
- Operating electrical seat adjustments
- Charging the wheelchairs batteries

There are many different control systems for wheelchairs on the market. If the controller on your wheelchair does not resemble the one in figure 1 contact your dealer.

DX Rem 48 by Dynamic Controls is a collective name for the entire control system of the wheelchair. The wheelchair is operated by means of a controller, which includes the following components:

Figure 1

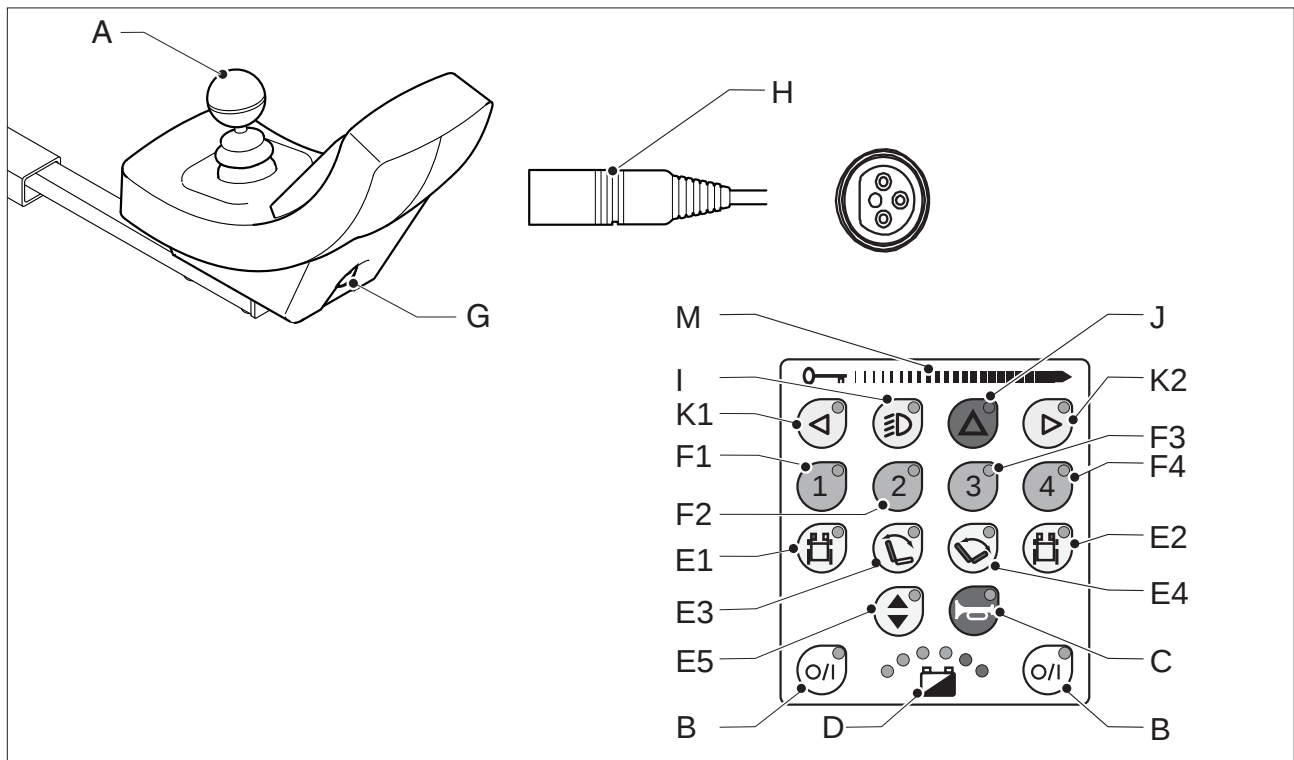
Figure 2: DX Rem 48

| Part                                   | Function                                                                                                                                                                                                                        |
|----------------------------------------|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| A. Joystick                            | In the 'drive' mode: driving and steering<br>In the 'adjustment options' mode: <ul style="list-style-type: none"> <li>Left/right to select the adjustment options</li> <li>Front/back to control the adjustment mode</li> </ul> |
| B. On/off switch                       | Switching the controller on or off                                                                                                                                                                                              |
| C. Horn                                | Warning signal with sound                                                                                                                                                                                                       |
| D. Battery indicator                   | Displays the power level of the battery                                                                                                                                                                                         |
| E.1 Adjustment option                  | Electrical adjustment of the left legrest (option)                                                                                                                                                                              |
| E.2 Adjustment option                  | Electrical adjustment of the right legrest (option)                                                                                                                                                                             |
| E.3 Adjustment option                  | Electrical adjustment of the angle of the backrest (option)                                                                                                                                                                     |
| E.4 Adjustment option                  | Electrical tilt adjustment of the seat (option)                                                                                                                                                                                 |
| E.5 Adjustment option                  | High/low seat adjustment (option)                                                                                                                                                                                               |
| F. 1 Speed regulator                   | Reduce driving speed (slowest)                                                                                                                                                                                                  |
| F. 2 Speed regulator                   | Increase driving speed (faster)                                                                                                                                                                                                 |
| F. 3 Speed regulator                   | Increase driving speed (faster)                                                                                                                                                                                                 |
| F. 4 Speed regulator                   | Increase driving speed (fastest)                                                                                                                                                                                                |
| G. Charge connector                    | Input for the battery charger                                                                                                                                                                                                   |
| H. Charge plug for the battery charger | Connector for the battery charger                                                                                                                                                                                               |
| I. Lights button                       | Switching the lights on or off                                                                                                                                                                                                  |
| J. Hazard lights                       | Warning signal with lights                                                                                                                                                                                                      |
| K.1 Direction indicator left           | Switches the left direction indicator on or off                                                                                                                                                                                 |
| K.2 Direction indicator right          | Switches the right direction indicator on or off                                                                                                                                                                                |
| M. Panel pad                           | Activate / deactivate the wheelchair by moving the magnetic key from left to right across the panel                                                                                                                             |

## **3 Driving the wheelchair with the controller**

### **3.1 Switching the controller on or off**

To be able to drive or operate the electric adjustment options of the wheelchair, the controller must be switched on. Press the on/off button (B in figure 2).

### **3.2 Driving the wheelchair**

Driving an electric wheelchair is done by operating a joystick. Move the joystick forwards and the wheelchair will also move forwards. Steer left and right and the wheelchair will turn.

### **3.3 Speed**

The maximum speed can be controlled by the speed regulator on the controller (F 1, 2, 3 and 4 in figure 2). Speed can be controlled with the joystick while driving. If the joystick is moved a little, the wheelchair will move more slowly.

## 4 Operating the electrical adjustment options

Not every wheelchair has been equipped with electric adjustment options. We make a distinction between four different adjustment options to the seating system:

- Tilt adjustments
- High/low adjustments
- Backrest adjustments
- Legrest adjustments

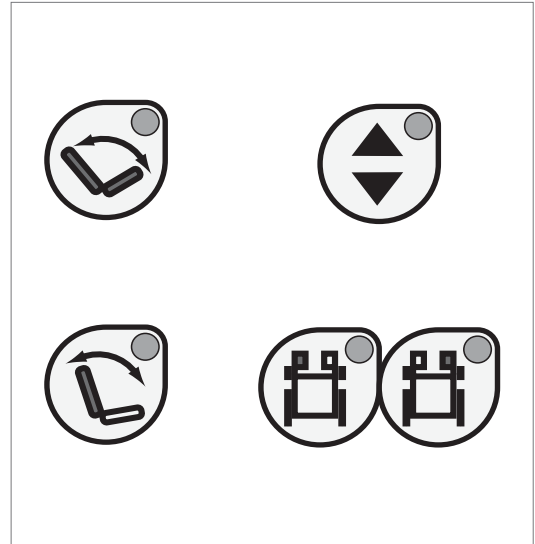
Figure 3

### 4.1 Selecting the desired adjustment option

1. Switch on the controller.
2. Press on the desired adjustment option (E in figure 2). The controller will now be in the 'adjustment options' mode. The LED light on the button indicates what electrical adjustment option is enabled (1, 2, 3, 4 or 5).
3. Moving the joystick forwards and/or backwards will activate the selected electric adjustment option (see table 4). Move the joystick forwards or backwards until the desired adjustment option has been attained.

|                             | Move joystick backwards                                       | Move joystick forwards                                         |
|-----------------------------|---------------------------------------------------------------|----------------------------------------------------------------|
| <b>Tilt adjustment</b>      | The entire chair will tilt backwards                          | The entire chair will tilt forwards                            |
| <b>High/low adjustments</b> | The entire chair will be raised                               | The entire chair will be lowered                               |
| <b>Backrest adjustments</b> | The backrest will tilt backwards                              | The backrest will tilt forwards                                |
| <b>Legrest adjustments</b>  | The legrest angle will increase, the footplate will be raised | The legrest angle will decrease, the footplate will be lowered |

Table 4

4. To return to the 'drive' mode: Press the selected adjustment option button again. The LED light of the speed button (F in figure 2) will light up. You are now to the 'drive' mode again.

*Note: If you adjust the seat height by using the high/low option, the speed will be reduced due to safety reasons.*

## 5 Lights

With the DX Rem 48 it is possible to control the following lights:

- Lights (I in figure 2)
- Hazard Lights (J in figure 2)
- Direction indicators (K1 & K2 in figure 2)

## 6. Troubleshooting

If the wheelchair will not function while the batteries are fully charged, check the following points before consulting your dealer:

- Switch the controller off and then switch it on again. Check to see if the malfunction has been solved.
- Check if the free wheel switch was switched to Drive.
- Check if the joystick was in the 0 position when the controller was switched on. In other words, the joystick must not be moved when the controller is being switched on or off.

### Malfunctions list

An extended malfunction list can be found in the appendix of the service manual (for qualified specialists only). The service manual can be found on [www.handicare.com](http://www.handicare.com)

## 7. Locking the controller

The Key Lock system uses a magnetic key to power the wheelchair down in such a way to prevent subsequent unauthorized power up and driving.

To arm the locking system, wipe the magnetic key from left to right on the path next to the key symbol on the DX Remote front panel. The system will beep and automatically power itself down and no LEDs should be lit.

To disarm the lock, switch on the remote and wipe the magnetic key from left to right on the path next to the key symbol on the DX Remote front panel. This will cause the LED's to stop flashing and the chair may now be driven normally. If the wheelchair is not disarmed within one minute, the remote will automatically turn itself off.

## 8 Technical specifications

|                                     |                                             |
|-------------------------------------|---------------------------------------------|
| <b>Operating voltage range:</b>     | 18V - 32V DC                                |
| <b>Charger rating:</b>              | 12A RMS Continuous, limited by DXBUS Rating |
| <b>Quiescent Current:</b>           | < 1mA Off, typically 100mA On               |
|                                     |                                             |
| <b>Operating forces</b>             | Less than 800 grams                         |
| <b>Protection rating:</b>           | IPx4                                        |
| <b>Operating Temperature Range:</b> | -25°C to +50°C                              |
| <b>Storage Temperature Range:</b>   | -40°C to +65°C                              |
| <b>Operating Humidity Range:</b>    | 0 to 90% RH                                 |

## 9. Technical diagrams

### 9.1 Technical diagram

The technical diagrams can also be found on the cover of the specific electronic component.

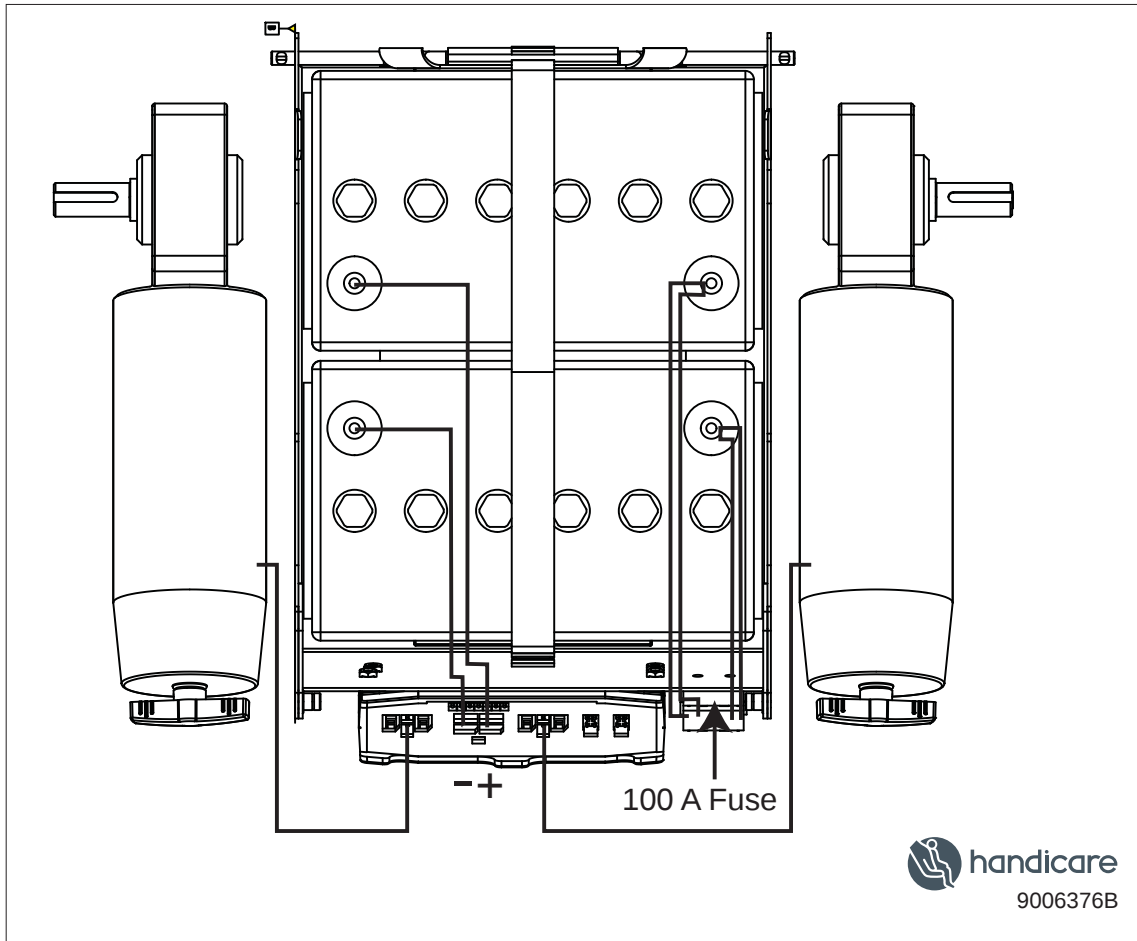
Figure 5

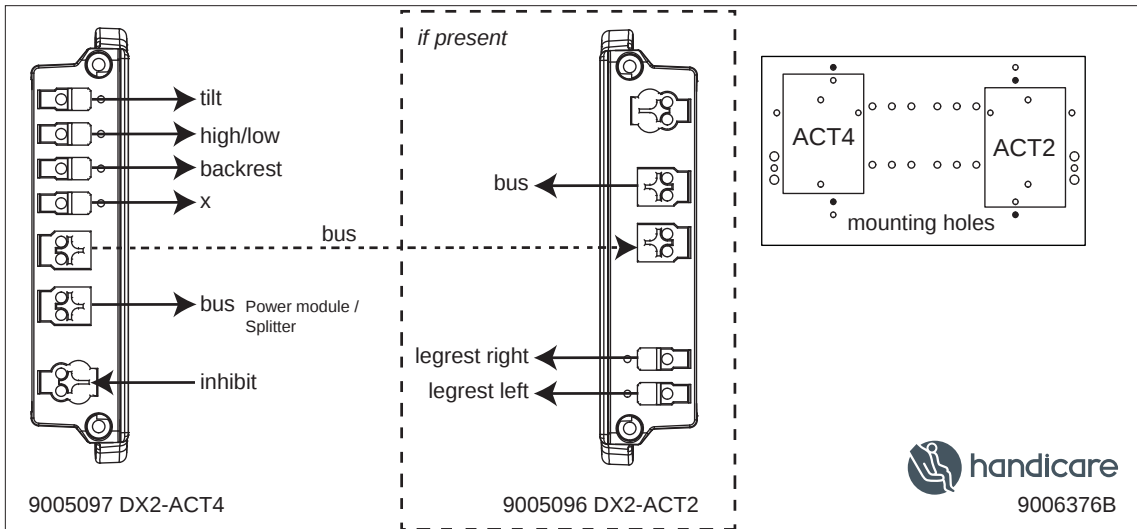
Figure 6

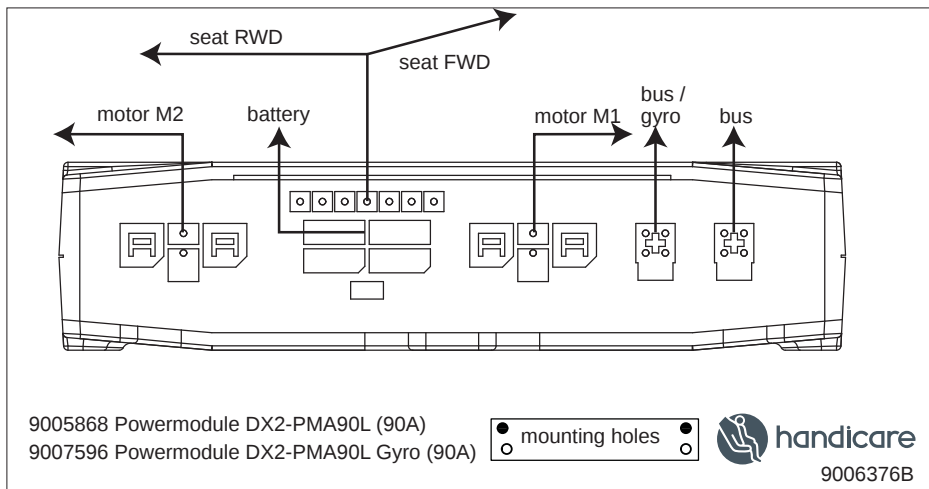
Figure 7

## 9.2 Technical diagram battery charger

The controller's standard configuration includes a '3-pin connection'. Ensure that the battery charger is properly connected so that the 'negative pole' and the 'inhibit' are connected, enabling the system to prevent the wheelchair from moving when the battery is being charged.

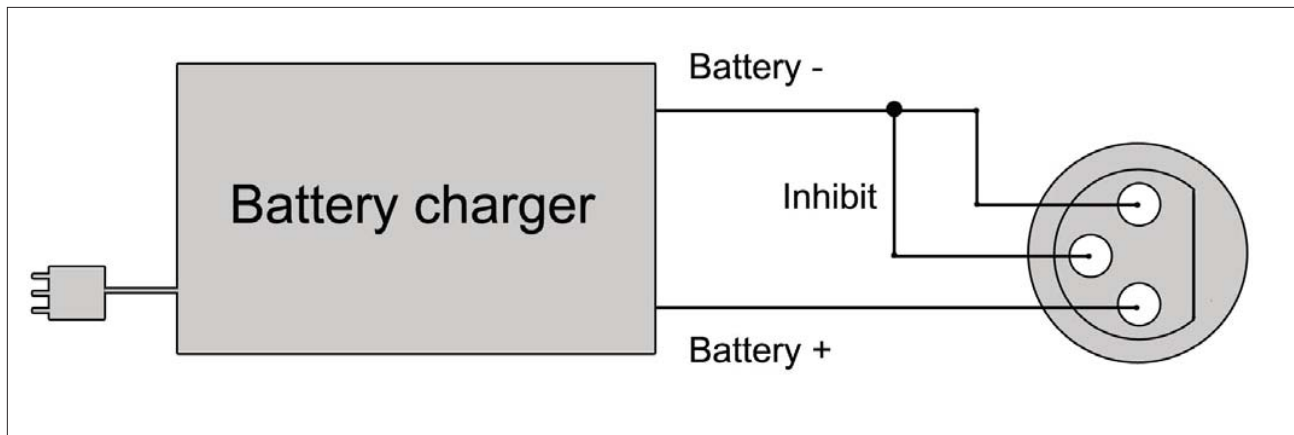
Figure 8

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Handicare B.V.